# cleanotix

SP Spray ótico

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EN Otic spray

For ear wax removal.

**About Ear Wax:** Wax or cerumen is a normal secretion of the glands in the ear. Excessive wax may cause a number of unwanted effects including hearing loss, discomfort/pain and coughing.

**Hearing loss** occurs when accumulated wax occludes (blocks) the ear canal, reducing or preventing sound waves from easily reaching the eardrum.

**Discomfort/Pain** may be evident in cases where compressed wax pressures the ear drum.

**Coughing** may be occurring in cases where the impacted wax stimulates nerves shared both by the ear canal and throat, provoking a "tickle in the throat" which may lead to cough.

## Ingredients:

Olive oil, olivaxol® and Spearmint Oil.

## Indication:

For the dissolution and removal of ear wax in the external ear canal.

**cleanotix** combines a mechanical wash compound with the safety and efficacy of natural paraffin and olive oils. Its use avoids the risk of perforation due to its equalized pressure and perfect dispersion. Earwax is dissolved and gradually removed from the ear within several hours/days (depending on the amount) by using a natural ear cleaning mechanism. Children from 2 years old.

## Warnings:

**cleanotix** should not be used in patients with perforated eardrum.  
Keep out of reach of children.  
Do not spray into the eye  
For external use only.

## Directions:

Turn applicator clockwise, Spray 1-2 puffs in each ear 3-5 times a day until the wax dissolves.

## Storage:

Store in a cool dark place. Do not refrigerate.  
From 4 °C to 50 °C.

Authorized representative: Obelis S.A.  
53 Boulevard Général Wahis  
1030 Brussels, Belgium

Distributor:  
Reva Health Europe S.L.  
Pl. Ausiàs March 1, 4º 6ª  
08195 Sant Cugat del Vallès, ESPAÑA  
info@reva-health.com

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Naveh Pharma Ltd