

Medical Device



## Nasal Gel

Prevents the penetration of viruses through the nose into the respiratory system

Net Wt. 20 ml



7 290017 847108 >

NAVEH PHARMA

**VirusStop\*** indicated for use as a protective mechanical barrier against allergens and viruses (e.g., SARS-CoV-2), intended for use to block inhaled viruses and bacteria within the nasal cavity.

**VirusStop\*** is a nasal gel consisting of hyaluronic acid and cellulose derivatives. When applied to the nostrils, the gel sticks to the walls of the nose, viruses that penetrate with the breath, stick to the gel.

The gel neutralize the virus by disrupting the outer layer of the virus, blocking viral growth and preventing the development of the flu.



OBELIS S.A.

53 Boulevard Général Wahis  
1030 Brussels, Belgium

Manufacturer: Naveh-Pharma Ltd.

P.O. Box 8139, Netanya 4250519, Israel.

Tel. 1-700-503-503 [www.navehpharma.co.il](http://www.navehpharma.co.il)



20 ml

Storage: Store in a cool, dry place, do not store in the refrigerator.

For external use only.

Warnings: Keep away from children. Avoid contact with eyes (if accidental eye contact occurs, rinse with water).



Instruction for Use: Apply 1 drop of **VirusStop\*** gel in each nostril. **VirusStop\*** is effective for several hours in preventing droplet infection, therefore, it is recommended to apply **VirusStop\*** always before entering an environment at high risk of infection, such as public-transport, airplane, sporting events, theaters, crowded places and before entering enclosed spaces or social gatherings people.  
Ingredients: Sodium Hyaluronate, Hydroxyethylcellulose, Propylen Glycol, Sodium EDTA, DL-Panthenol, Allantoin, Sodium Chloride, Glycerin, Purified water.



Prevents the penetration of viruses through the nose into the respiratory system

## Nasal Gel



Prevents the penetration of viruses through the nose into the respiratory system

## NASAL GEL

