



For Ear Wax Removal

About CleanEars

Ear wax or cerumen is a normal secretion of the glands in the ear. Excessive wax may cause a number of unwanted effects including hearing loss, discomfort, pain and coughing.

Hearing loss occurs when accumulated wax occludes (blocks) the ear canal, reducing or preventing sound waves from easily reaching the eardrum.

Discomfort or pain may be evident in cases where compressed wax pressures the ear drum.

Coughing may be occurring in cases where the impacted wax stimulates nerves shared both by the ear canal and throat, provoking a «tickle in the throat» which may lead to cough.

CleanEars is an effective safe & natural spray intended to dissolve and wash away accumulated ear wax. **CleanEars** combines a mechanical wash compound with the safety and efficacy of natural olive oil. Using **CleanEars** earwax is dissolved and gradually removed from the ear by using a natural ear cleaning mechanism. **CleanEars** spray mechanism ensures perfect dispersion in the ear canal.

Ingredients: Mineral Oil,
Phytosqualan, Spearmint Oil.



Indication: For the dissolution and removal of ear wax in the external ear canal for babies over 3 month, children and adults.

Warnings:

- **CleanEars** should not be used in patients with perforated eardrum
- Keep out of the reach of children
- Do not spray into the eye

FOR EXTERNAL USE ONLY

Directions: Spray one puff in each ear 3-5 times a day until the wax dissolves.

In difficult cases straight the ear canal by pulling the upper ear lobe slightly upward and toward the back of your head. This maneuver allows penetration of the active ingredients to the inner parts of the ear canal.

Storage Conditions: Store in a cool place, do not expose to temperature over 50°C.

Managing empty spray containers: **CleanEars** containers are exempt from hazardous awaste rules. They have no special storage, labeling or disposal requirements. Recycle them, if possible or send them to an incinerator that will recover the metal.



Naveh-Pharma LTD.

11, Yad-Harutzim St., Netanya, 4250519 Israel

www.navehpharma.com



OBELIS S.A. 53 Boulevard

Général Wahis 1030 Brussels, Belgium

The lifetime of the device 36 months from the manufacturing as stated on the label

Version 3 5/2021